



# CLEANING THE LEVL DEVICE

---

**WARNING:** Failure to follow the proper cleaning protocol may cause inaccurate measurements, the sensor to burn out, or damage to the internal components of the LEVL device. Note: Alcohol cleansers will damage the sensor.

**Please use the approved disinfectant wipe (listed below) to disinfect the LEVL device**

Approved cleaner : Checkers EPA-Certified Bleach Based Disinfectant Wipe

**LEVL breath pods (perform after each use)**

Cleaning protocol

- Pull apart the breath pod base and cap.
- Wipe the inside and outside of each part.
- Allow to air dry for 10 minutes in a disinfected workspace.
- Reassemble the breath pod and store in a sanitary place until its next use.

**LEVL device exterior and under the lid (perform after each use)**

Cleaning protocol

- Wipe down the top of the lid, sides, front and back of the LEVL device.
- Wipe down the underside of the lid.
- Wipe storage port and the testing port.
- Wipe all other areas under the lid.
- Allow 10 minutes to dry before using the LEVL device.